


IMPLADENT LTD

REGENERATIVE SOLUTIONS

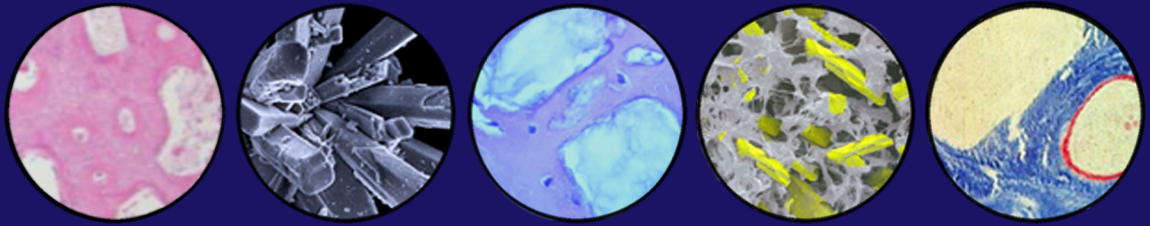

REGENERATIVE PRODUCT OVERVIEW



Order online at www.impladentltd.com or call (800) 526-9343

OSTEOGEN®

NON-CERAMIC BIOACTIVE RESORBABLE CALCIUM APATITE BONE GRAFTING MATERIAL



Size	Price
0.75 g / 1.5 cc	\$69.00
1.5 g / 3.0 cc	\$99.00
3.0 g / 6.0 cc	\$149.00

- The Non-Ceramic Bioactive Resorbable Calcium Phosphate crystals and crystal clusters are physicochemically and crystallographically similar to human trabecular bone
- Unique crystal cluster architecture intertwines to form 360° lattice assuring ample porosity for host cell proliferation and integration
- Crystal clusters are radiolucent on day of surgery and radiopaque in 4-6 months following resorption via osteoclast cells and host bone formation
- The OsteoGen® production process yields a unique Ca:P ratio that is NOT a β -TCP and NOT a dense ceramic HA, nor is it a biphasic mixture of the two
- Safe, reliable and effective for use with implants for over 35 years

After tooth extraction,
remove all root fragments




Debride and make holes
through the Lamina Dura



Compact OsteoGen® well
Product shows radiolucent on
the day of surgery



At ~6 months on average, product
shows radiopaque as resorbing
crystals convert to bone

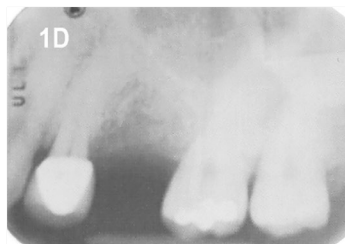

OsteoGen® Bioactive Resorbable
Crystals are ideal for implant
placement



Photos Courtesy of
E. Richard Hughes, DDS

OSTEOGEN® SYRINGES

BIOACTIVE RESORBABLE CALCIUM APATITE PRE-FILLED SYRINGES

- OsteoGen® also available in convenient 0.3g pre-filled syringes
- Each box contains six pre-filled syringes

Size	Price
0.3g Pre-Filled Syringes (Box of 6)	\$234.00



OSTEOGEN® BONE GRAFTING PLUGS

ONE STEP BONE GRAFTING SOLUTION FOR SOCKET PRESERVATION NO MEMBRANE NEEDED FOR SOCKET GRAFTING


- OsteoGen® Bone Grafting Plugs are an easy and affordable way to graft extraction sites
- Mineral and Collagen composition mimics structure of human bone



**OSTEOGEN®
NON-CERAMIC
BIOACTIVE CRYSTAL
BONE GRAFT**

**•TYPE I BOVINE
ACHILLES TENDON
COLLAGEN**


Size		Price
Large: 10mm D x 20mm L	Box of 5	\$279.00
	Box of 10	\$499.00
Slim: 6mm D x 25mm L	Box of 5	\$279.00
	Box of 10	\$499.00
Extra Large: 15mm D x 20mm L	Box of 5	\$499.00

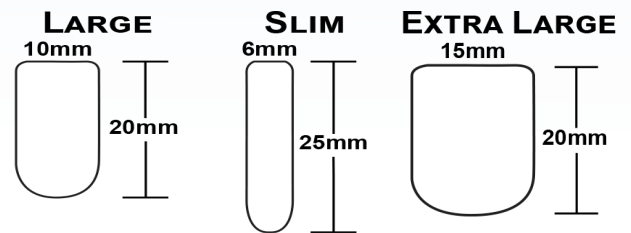

OSTEOGEN® STRIP

RESORBABLE MINERALIZED COLLAGEN BONE GRAFT

Size	Price
20 x 40 x 3 mm (Box of 2)	\$199.00
12 x 20 x 3mm (Box of 2)	\$99.00

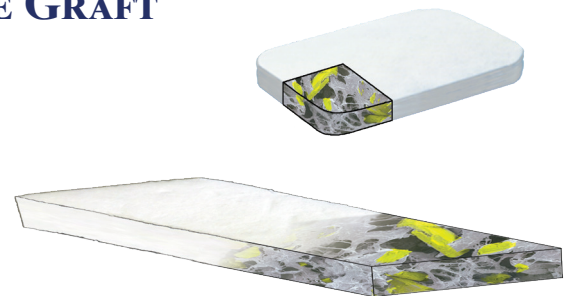


- Flexible strips easily conform to grafting site, eliminating gaps in irregular bony walls
- Eliminates particulate migration issues, especially in the sinus
- Used next to immediately placed implants in extraction sites to fill any potential gaps between the implant and socket wall
- Can be folded or layered multiple times to create additional volume and is easily packed into the defect space
- Extremely porous and hydrophilic. Will soak up endogenous blood, or can be used as a mineralized scaffold for added growth factors or serum from platelet concentrates

OSTEOGEN[®] MALLEABLE BONE BLOCK RESORBABLE MINERALIZED COLLAGEN BLOCK

- The Malleable OsteoGen[®] Bone Block is used in onlay grafting procedures together with the TriStar[™] Screws and is easily adaptable to the contours of defect site.
- OsteoGen[®] is a bioactive and resorbable calcium apatite crystal graft that is physicochemically and crystallographically similar to human bone.
- The Bovine Achilles Tendon collagen carries the graft for easy and efficient delivery to the site, eliminating the hassle of packing and securing particulate grafts for large defects.
- The Type I collagen absorbs and deliver blood flow to the slowly resorbing graft, a feature critical for the initiation of formation and early angiogenesis.

Size	Price
10x15x17mm Box of 2	\$219.00

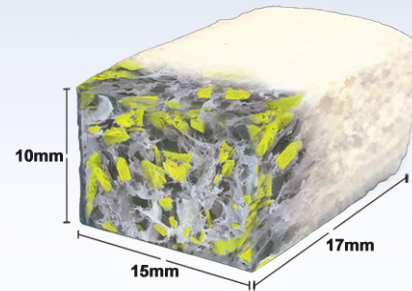

OSTEOGEN[®] DENSE PELLETS

- OsteoGen[®] Pellets should be mixed with OsteoGen[®] crystals and the patient's blood; add collagen for larger defects
- Used to build up or provide bulk density mass for pontic ridge height
- Designed for load bearing applications; see our website for additional clinical suggestions



Size	Price
1.0 g Vials Pellets 1.3mm H x 2.2mm D	\$59.00

OSTEOGEN PLUG PRESSERSM


- Used to compress and add density to OsteoGen[®] Bone Grafting Plugs for reinforcing and rebuilding the buccal plate.
- Can be combined with OsteoGen[®] Dense Pellets for increased support.
- Pressing and twisting motion can be used to morselize pellets, allograft or autogenous bone.
- Surgical Stainless Steel with a "Black Diamond Finish"



Size	Price
49mm	\$199.99

HUMAN BONE ALLOGRAFTS

NonDemin™

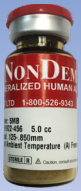

Mineralized Cortical Allograft

Size	Particle Size	Price
0.5 cc	.125-.850 mm	\$73.00
1.0 cc	.125-.850 mm	\$88.00
2.0 cc	.125-.850 mm	\$119.00
5.0 cc	.125-.850 mm	\$285.00

Corticocancellous “50/50” Blend



Mineralized Corticocancellous

Size	Particle Size	Price
1.0 cc	.250-1.00 mm	\$102.00
2.0 cc	.250-1.00 mm	\$134.00
5.0 cc	.250-1.00 mm	\$305.00

OsteoDemin™



Demineralized Cortical Allograft

Size	Particle Size	Price
0.5 cc	.125-.850 mm	\$73.00
1.0 cc	.125-.850 mm	\$88.00
2.0 cc	.125-.850 mm	\$119.00
5.0 cc	.125-.850 mm	\$285.00

CancelOss™



Mineralized Cancellous Bone Allograft

Size	Particle Size	Price
1.0 cc	.500 - 1.00 mm	\$102.00
2.0 cc	.500 - 1.00 mm	\$134.00
5.0 cc	.500 - 1.00 mm	\$305.00

CurOss™



Mineralized Large Particle Cortical

Size	Particle Size	Price
1.0 cc	~1.0 - 2.0 mm	\$120.00
2.0 cc	~1.0 - 2.0 mm	\$180.00

ULTRA-COARSE DIAMOND BUR KIT

- Debride - Ultra Coarse Diamond (UCD) Debridement Burs are specifically engineered to quickly and effectively grab and remove granulation tissue following tooth extraction.
- Decorticate - Make multiple “bleeders” using the included round carbide bur by perforating the lamina dura into the alveolar process to trigger the Regional Acceleratory Phenomenon.
- Graft - Deliver the bioactive OsteoGen® Bone Grafting Plug into the bleeding socket, compress and suture over top - no additional membrane required.

Size	Price
1.0 mm UCD Bur	\$179.00
2.5 mm UCD Bur	
3.5 mm UCD Bur	
#6 Round Bur	
Bur Holder	





COLLAFORM® PLUGS & SINGLES

- Resorbable bovine collagen wound dressings are economical and easy to handle
- Resorption time is 4-6 weeks on average
- Achilles Tendon matrix maintains graft in extraction site for keratinized tissue restoration



Size	Price
10mm x 20mm (Box of 5)	\$70.00

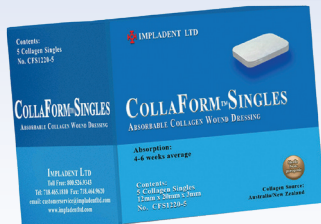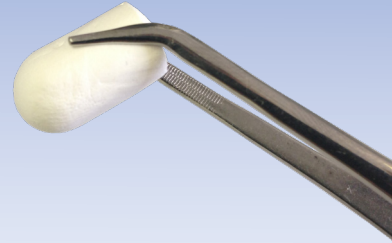

Size	Price
12 x 20 x 3mm (Box of 5)	\$185.00



OSTEOGEN® PLUG STAINLESS STEEL PLUGGER

Size	Price
6mm/3mm	\$36.00



OSTEOGEN® PLUG SERRATED CURETTE

Size	Price
2.4mm	\$36.00
3.7mm	\$36.00



HELI MEND® ADVANCED

- Bovine collagen membrane with strong and predictable resorption
- Extended resorption time of ~18 weeks (4-5 months) on average
- Derived from purified Achilles Tendon yielding excellent handling characteristics



Size	Price
15mm x 20mm	\$99.99
20mm x 30mm	\$129.99
30mm x 40mm	\$164.99



TRISTAR™ BONE GRAFT FIXATION SYSTEM

STARTER KIT: 15 SCREWS, TITANIUM MESH AND BONE GRAFT

- Kit includes 15 screws up to 14mm in length, titanium mesh, 2 vials of bone graft, handle, driver & sterilization cassette
- SQUARELOCK™ CONNECTION ensures an ultra-secure fit using only one tool
- Partially threaded & tapered design on screws above 5mm can be used to secure **BONE BLOCKS** or used as **ADJUSTABLE TENTING SCREWS**
- 0.1mm thick titanium mesh has no memory
- One large sheet of titanium mesh (40x60mm) or 3 smaller ones (18x25mm) included
- 0.75g (1.5cc) OsteoGen® and 1.0cc NonDemin™ Mineralized Cortical Bone Allograft included
- Screw lengths: 3mm, 4mm, 5mm, 6mm, 8mm, 10mm, 12mm, 14mm

~~RETAIL PRICE~~

~~\$1,558~~

PRICE AS KIT

\$999



AVAILABLE SCREWS:

STARTER KIT INCLUDES:

- Choice of 15 Screws (3 Packs of 5: 3mm - 14mm)
- Sterilization Cassette
- SquareLock™ Driver
- Spring Loaded Handle
- Choice of Ti-Mesh: (1) 40x60x0.1mm or (3) 18x25x0.1mm
- 0.75g OsteoGen® Bioactive Resorbable Calcium Apatite
- 1.0cc NonDemin™ Mineralized Cortical Bone Allograft

LENGTH	THREADED / UNTHREADED
3mm	Fully Threaded
4mm	Fully Threaded
5mm	Fully Threaded
6mm	4mm / 2mm
8mm	5mm / 3mm
10mm	5mm / 5mm
12mm	6mm / 6mm
14mm	7mm / 7mm

IMPLADENT LTD

REGENERATIVE SOLUTIONS

OsteoGen®, TriStar®, CollaForm® and CurOss® are registered trademarks of Impladent Ltd.
HeliMend is a registered trademark of Integra LifeSciences Corporation