

## *Partial pulpotomy of anterior teeth #1*



Partial pulpotomy performed using ENDOCER Zr  
and resin filling restoration on a fractured anterior tooth  
Unlike conventional MTA cases,  
there is no dark color observed underneath the restoration.



## ENDOCER Zr Case Report

Mineral Trioxide Aggregate

### *Partial pulpotomy of anterior tooth #2*



Due to a pin-point pulp exposure after preparation, partial pulpotomy was performed to ensure thickness of ENDOCER Zr and resin filling. After partial pulpotomy and bleeding controlled with NaOCl, ENDOCER Zr was used to fill and condensed well with cotton ball to ensure proper sealing. ( Excessive force should be avoided.) After confirmation of setting of ENDOCER Zr, resin filling was completed for protection after removing a slight amount of ENDOCER Zr in cavity opening.