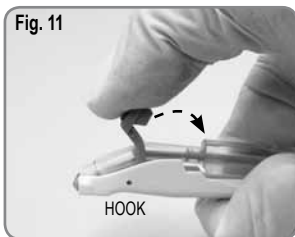
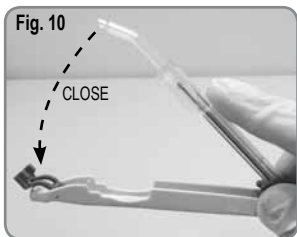
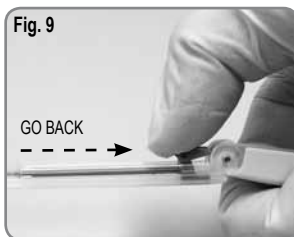
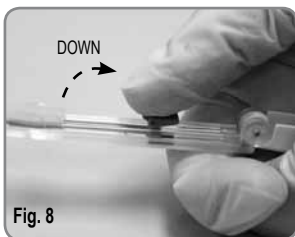
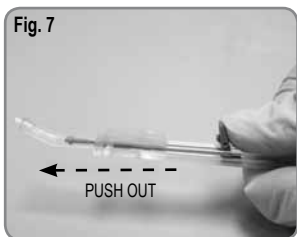
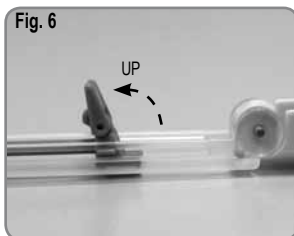
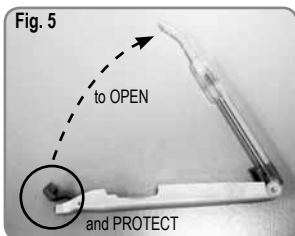
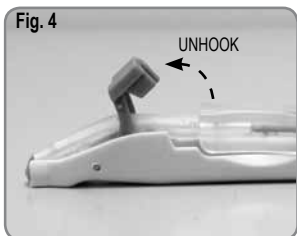
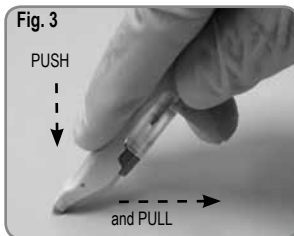
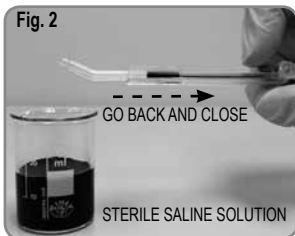
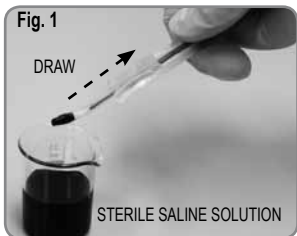
SMARTSCRAPER

CE
0123

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Intended use

The vast experience of META in the harvesting of cortical bone has resulted in the development of Smartscraper, a tool to harvest autologous bone for direct in-situ grafting as part of oral surgery applications.

- Before use, check that the package is intact.
- Do not use the product if the package is damaged.
- The Smartscraper device must be used exclusively by skilled medical staff.
- The surgeon must assess the suitability of patients for bone grafting and the appropriate surgical procedure.
- The surgeon must take all necessary precautions to prevent the risk of infection and complications associated with bone grafting.
- The device can only be used once. Do NOT clean or sterilise the device after use.
- **Warning:** if it is reused on another patient, there is a risk of cross-contamination and loss of performance and functional characteristics of the device.
- Always wear sterile gloves when handling the Smartscraper device and strictly follow the procedures in order to guarantee a sterile environment. Make sure you do not cut sterile gloves with the sharp end of the tool.
- Discard after use in special medical waste containers in compliance with the regulations in force.
- Meta does not shoulder any responsibility for improper use of the product.

- 1 Open the package from the side showing the symbol .
- 2 The device is fitted with a green piston to draw a sterile saline solution before bone harvesting begins (see Figs.1 and 2). This makes it easier to harvest the bone and keep it hydrated.

Harvest the bone by pushing the end of the device towards the surface of the bone and then pull the device back at the same time (see Fig. 3). Bone can also

be harvested by making oblique and sideways movements. Vary the angle of the tool and the pressure applied to obtain the required consistency of bone chips or shavings. The Smartscraper reservoir can contain up to 0.30 cc of bone.

- 3 The tool must be held with the transparent reservoir always facing towards the bone. Hold the device securely to ensure precise movements during use (see Fig. 3).
- 4 Smartscraper is designed with a practical syringe allowing the harvested bone to be grafted directly onto the site of the implant or defect. Follow the instructions shown in Figures 4 to 7 to extract the syringe and prepare the piston for thrusting. Release the safety lever (see Fig. 4) to detach the container from the blade holder body. The device is also designed to protect the user from accidental contact with the sharp blade (see Fig. 5). Proceed to filling the bone defect by injecting the harvested bone directly in situ. (Fig. 7).
- 5 Close the device and follow the instructions in Figures 8 to 12 to continue the bone harvesting and grafting procedure.
- 6 Set the device to the closed and locked position (see Figures 10 and 11) before disposing of it.

Useful tips:

In the event of poor cutting quality, adjust the angle of the device and the pressure exerted on the bone.
If this does not provide improved performance then the blade may be worn. Change the device.

These instructions for use as well as further information about the tool are available on the website: www.metahosp.com

Please take the time to read this information to prevent any errors and ensure good operation of the product.

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**C.G.M. S.p.A.**