

IMPLASEAL

ImplaSeal™ instructions for use

The Implaseal™ product is designed to solve the problems of what to fill the implant abutment or prosthetic screw channel with. For years, dentists have used cotton pellets or pieces of plumber's tape with little hygienic success. This product is designed to solve that very issue by providing a simple, yet effective antibacterial and antifungal solution in a great smelling formula embedded on the foam sphere to simplify the insertion and more importantly, to resist the proliferation of the pathogens known to harbor in those areas.

How supplied:

The product of 2 foam beads come embedded in a proprietary formula of antibacterial and antifungal agents suspended in eucalyptus and coconut MCT oil for a safe and refreshing solution.

Application instructions:

With simplicity being a factor, we have designed the perfect size of approximately 5 mm foam spheres that a provider simply picks up with everyday cotton pliers and places into the abutment or screw channel of choice. Once the sphere is inserted, a more definitive final restorative solution of composite can be bonded over the orifice.

****Does not need any curing or waiting. Just insert and restore.***

